

Preserving the life of your unit

Dürr Dental invented the modern day suction system so have a wealth of experience to draw on when it comes to maintenance. To preserve the life of your unit they offer the following tips (based on over 50 years of experience):



1. Aspirate at least one glass of water through the spittoon and the suction hose after every patient to remove blood, saliva and dentine residue
2. Use cold water to mix disinfectant as hot water inactivates many disinfectant components and tends to create foam and cause coagulation when combined with blood
3. Just as with household appliances, limescale can damage components. Depending on water hardness, use MD555 cleaner at least once a week
4. Use only foam-free products intended for the job – never use household cleaning agents in the suction unit
5. Never mix products as this can neutralise the disinfectants
6. Do not use the suction unit to vacuum drawers!
7. Carry out the recommended maintenance.

Dürr Dental are the only supplier of suction systems who also manufacturer cleaning solution. Their Orotol range is the leading brand of suction disinfectant in the world. It boasts an extraordinary cleaning power and is foam-free making it popular with dental nurses. The suction range is available as either a concentrated powder or as a liquid, both of which have an environmentally friendly composition and a pleasant odour.

For more information call 01536 526740.

Orthodontic and brace care


The common denominator of second teeth and the hormonal changes of adolescence is the depletion of hyaluronic acid, the normal constituent of healthy periodontal tissues. The consequence of this is a reduction in the level of protection given to the mucous membranes of the oral cavity. Gengigel Gel Teen has been formulated to adhere to the mucosa, allowing high molecular weight hyaluronan to penetrate the gums below and restore a healthy balance.



Gengigel is also ideal for children wearing braces, who are more susceptible to gum inflammation, ulcers and lesions caused by rubbing of the appliance. The gel is designed to be massaged gently onto affected areas where it is clinically proven to alleviate pain and rapidly repair damaged gums.

For more information call 0208 459 7550 or email marketing@dentocare.co.uk.

A dynamic and interactive training campus

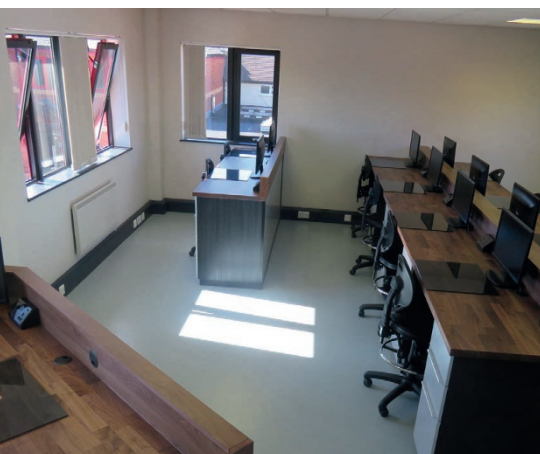

As one of the world's most innovative dental companies, GC has continuously made important and ground breaking contributions to dental care. This is coupled with their philosophy to promote communication and understanding between dentists and technicians, with the aim of bringing new concepts, products and methodology to the market.

To enhance this philosophy even more GC UK has opened a purpose built campus at their head office in Newport Pagnell. The Campus incorporates ten training workstations and one instructor position each with its own localised monitor to facilitate

personalised training in a light, airy and calm atmosphere. Ideal for trainee participation, it provides a dynamic and interactive learning environment which can easily be adapted for dentist, technician or nurses' courses using a wide range of GC's market leading products.

GC UK look forward to welcoming dentists, technicians and nurses from throughout the UK to one of their courses soon.

For further information contact GC UK Ltd on 01908 218999, email info@gcukltd.co.uk or visit www.gceurope.com.